

FROM THE #1 LEADER
IN OVERDENTURE ATTACHMENTS AND CREATOR OF LOCATOR



CHAIRSIDE®

Attachment Processing Material



MAKE IT A LOCATOR LIFE

CHAIRSIDE® is a unique **Attachment Processing Material** from Zest Dental Solutions, the manufacturer of the market-leading LOCATOR® overdenture products for more than 40 years. CHAIRSIDE Attachment Processing Material was designed for ease of use and predictability when processing attachments into full and partial overdentures, using either a chairside or laboratory procedure.

Clinician input contributed to formulations that allow clinicians to carry out their procedures the way they want to, easily, quickly, and at a reduced cost per case*.

CHAIRSIDE Attachment Processing Material Key Attributes:

SIMPLE

- No primer required for reduced chair time
- Bonds to itself for easy void-fills and touch-ups
- Dual-cure capability provides convenience and flexibility

VERSATILE

- Normal Set and Fast Set varieties match the complexity and speed of your procedure
- Available in 8g syringes or 18mL cartridges
- Shorter mixing tips and ideal viscosity for precise application, as well as reduced waste and cost

PATIENT FRIENDLY

- Low curing temperature
- Low odor and minimal taste



Dr. Michael Scherer, DMD, MS

CHAIRSIDE Attachment Processing Material by Zest is a game-changer! Zest has developed a material that is perfect for processing attachments...It's so easy to use, no messy primer is required, and the material offers dual-curing flexibility (self or light), thereby making the time I spend connecting the denture caps to the overdenture highly efficient and effective.

*Compared to the leading brands.

THE BEST PICK-UP MATERIAL COMES FROM ZEST!

Every practice, clinician, and case is different. That's why CHAIRSIDE Attachment Processing Material is now available in Normal Set and Fast Set varieties, allowing the clinician to pick the material that best suits their experience and needs.



Working Time
at Room Temperature
(73°F/23°C) min:sec

1:45

0:30

Set Time
at Room Temperature
(73°F/23°C) min:sec

5:00

2:00

Best For:

- Designed for extended working time and flexibility.
- Universal indications for all overdenture attachment procedures.
- Recommended for intraoral procedure of any implant number.
- Products: LOCATOR, LOCATOR R-Tx.

- Designed for accelerated set and efficiency.
- Laboratory or intraoral procedures of up to 3 housings per application.
- Luting together seperated components chairside that need to be soldered or welded.
- Verification jigs.
- Products: LOCATOR, LOCATOR R-Tx.

PRODUCT ORDERING INFORMATION

DESCRIPTION	NORMAL SET ITEM NUMBER	FAST SET ITEM NUMBER
CHAIRSIDE® Attachment Processing Material 8g Syringe (15 brown mixing tips and 15 angled tips included in each package)	09566	90-00130
CHAIRSIDE® Attachment Processing Material 18mL Cartridge (10 blue mixing tips included in each package)	09565	90-00132
CHAIRSIDE® Attachment Processing Material Brown Mixing Tips (15 mixing tips to use with 8g syringe)	09125-15	
CHAIRSIDE® Attachment Processing Material Blue Mixing Tips (15 mixing tips to use with 18mL cartridge)	09973-15	



8g Syringe and Plunger



18mL Cartridge



Mixing Tips



Angled Tips
(15)



Mixing Tips

CONTACT US WITH **ZEST LIVE!**™



REAL-TIME
SUPPORT WHEN
AND WHERE YOU
NEED IT.

Visit our website
WWW.ZESTDENT.COM
for video, voice, or text
chat support LIVE.



Zest Dental Solutions
2875 Loker Avenue East
Carlsbad, CA 92010 USA

Direct: 1.800.262.2310
Fax: 1.760.444.4881
Email: sales@zestdent.com

©2022 ZEST Anchors LLC. Bulk EZ, Cem EZ, IntelliTek, Iveri, CHAIRSIDE, ZEST and Zest Dental Solutions are registered trademarks of ZEST IP Holdings, LLC. Some products may not be regulatory cleared/released for sale in all markets. Please contact the local Zest Dental Solutions representative for current product assortment and availability. All trademarks and registered trademarks are the property of their respective owners.
*Data on File, Zest Dental Solutions

L9202 REV K 05/22