




CeraGlaze™ Porcelain Polishing Set

LS-904

Kerr™

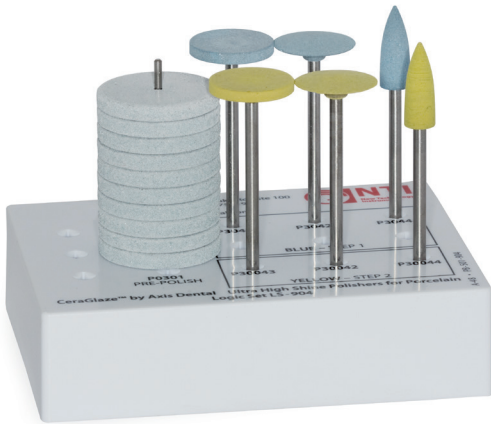

This Logic Set contains the essential polishers for finishing porcelain. No need for polishing paste or reglazing. Delivered in a plastic block.

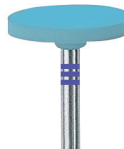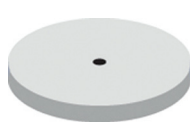

HEAD Ø 1/10 mm	220	055	150	145	055	150	145
LENGTH (mm)	3.0	15.5	1.0	2.5	15.5	1.0	2.5
HP shank - 1 pk	P3001-220 10-pk	P3044	P3042	P3043	P30044	P30042	P30043

CeraGlaze is a remarkable diamond impregnated polishing system used to restore a natural high shine to porcelain and eliminating the need for re-firing. Autoclavable. Handpiece sold individually. Friction grip and RA/latch sold in packs of 3. HP & RA latch recommended speed: 5000-15,000 rpm. FG polishers at 10,000-12,000 rpm and with light pressure.



Green Adjusting:

RPM: 15,000 Max
Pressure: Light for adjusting
Water: None during this step.



Blue Refinement:

RPM: 10,000 – 12,000 Max
Pressure: Light - Medium
Water: Enough to create a slurry
Time: 45 seconds to create a dull luster underneath a small layer of film.



Yellow High Shine:

RPM: 5,000 – 7,000 Max
Pressure: Medium
Water: Enough to create a slurry
Time: One minute to create a wet look or until desired effect.