

OPTIGLAZE™ color

A LIGHT-CURED CHARACTERIZATION COATING FOR COMPOSITE AND ACRYLIC INDIRECT RESTORATIONS

GC

WELCOME TO OPTIGLAZE™ color

A LIGHT-CURED CHARACTERIZATION COATING DESIGNED FOR COMPOSITE AND ACRYLIC INDIRECT RESTORATIONS.

OPTIGLAZE color is a revolutionary new material developed specifically to stain and glaze your CERAS^{SMART} milled blocks, composite, acrylic, and PMMA type materials. You can now change shades, shift value, characterize, create effects and glaze all your resin type prosthetics in no time at all. Best of all, offer your dentists and patients a more beautiful and aesthetic restoration that will last due to GC's innovative new homogeneously dispersed nano filler technology.



EVERYTHING LOOKS **BETTER** IN COLOR



ARE YOU LOOKING FOR A SIMPLE SOLUTION? ADD CHARACTERIZATION TO YOUR EVERYDAY AESTHETIC CREATIONS.

The versatility of OPTIGLAZE color will impress you because it adds color, surface gloss, and high wear resistance to all your needs:

- Composite crowns and bridges
- Temporary resin crowns and bridges
- Hybrid ceramic crowns
- Implant retained dentures
- Acrylic permanently fixed implant bridges
- Artificial teeth
- Acrylic dentures



Restorations by Christian Rothe MDT, Germany



Restoration by Joshua Polansky, United States

OPTIGLAZE™ color THE COMPLETE SET




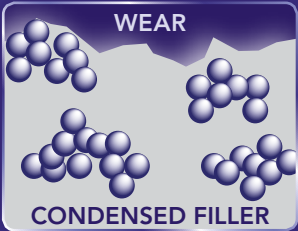
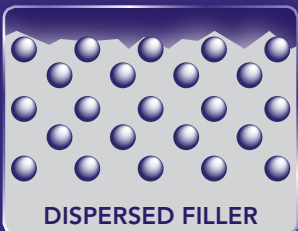
NANO-FILLER TECHNOLOGY

OPTIGLAZE™ COLOR WILL IMPRESS YOU

Ready to use, easy-to-handle, and cuts on the polishing stage, saving you valuable time.

- Can choose from a wide variety of colors, both for internal and external characterization, leading to perfect aesthetics.
- Renowned nano-filler technology gives your restoration high-wear resistance, for long-lasting gloss with a high discoloration resistance

OPTIGLAZE color contains GC's renowned nano-filler technology, giving it a high wear resistance for long-lasting gloss.

| CURRENT TECHNOLOGY | | NANO-FILLER TECHNOLOGY |
|--|---|--|
|  <p>WEAR</p> <p>WITHOUT FILLER</p> |  <p>WEAR</p> <p>CONDENSED FILLER</p> |  <p>DISPERSED FILLER</p> |
| <p>Without filler, the material can wear easily.</p> | <p>With conventional filler technology, the clustering of the particles readily occurs and the material can still be susceptible to wear.</p> | <p>Due to GC's new nano-filler technology, the particles are evenly dispersed, offering a high wear and discoloration durability and a long lasting gloss!</p> |

Legend:

- Wear
- Nano-filler
- Resin matrix of the coating agent



FEATURES & BENEFITS

TIME SAVING

- Ready-to-use
- Light-curing cuts polishing stage
- Easy handling and application

PERFECT AESTHETICS

- A wide variety of colors available
- Long-lasting gloss
- Thin coating layer (25-50 μm)

DURABLE RESULTS

- High wear resistance
- Excellent bonding durability to a wide variety of polymer-based materials
- High discoloration resistance



Restorations by Philippe Llobell MDT, France



Restorations by Jean François Deche MDT, France

CLINICAL IMAGES



Restorations by Roeland DePaepe MDT, Belgium.



Restorations by Nikos Bellas MDT, Greece.



Restorations by Simone Maffei MDT, Italy.

TO GLAZE ONLY, USE OPTIGLAZE™




OPTIGLAZE™

FOR AN AESTHETIC GLOSSY FINISH.

For a simplified finishing process, GC has developed a new coating agent called OPTIGLAZE, which provides an aesthetic glossy surface on artificial teeth, removable dentures, temporary crowns and individual acrylic trays. It can be used in difficult to polish areas, such as posterior fissures or interproximal areas of indirect composite restorations. So, instead of mechanical polishing, take the easier option and get a higher quality final gloss with OPTIGLAZE.

BENEFITS

- Time saving
- Easy handling
- Perfect aesthetics
- Long lasting gloss
- New nano-filler technology
- Superior wear resistance
- Excellent bonding durability
- Very thin coating layer (25-50 µm)
- High discoloration resistance
- No influence on restoration color

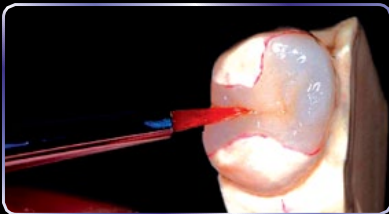

PRODUCT SKUs



OPTIGLAZE™ color

008408 OPTIGLAZE color, Set

008409 OPTIGLAZE color A Plus, 2.6mL

008410 OPTIGLAZE color B Plus, 2.6mL

008411 OPTIGLAZE color C Plus, 2.6mL

008412 OPTIGLAZE color White, 2.6mL

008413 OPTIGLAZE color Ivory White, 2.6mL

008414 OPTIGLAZE color Yellow, 2.6mL

008415 OPTIGLAZE color Orange, 2.6mL

008416 OPTIGLAZE color Pink Orange, 2.6mL

008417 OPTIGLAZE color Red Brown, 2.6mL

008418 OPTIGLAZE color Olive, 2.6mL

008419 OPTIGLAZE color Grey, 2.6mL

008420 OPTIGLAZE color Blue, 2.6mL

008421 OPTIGLAZE color Lavender, 2.6mL

008422 OPTIGLAZE color Pink, 2.6mL

008423 OPTIGLAZE color Red, 2.6mL

008424 OPTIGLAZE color Clear, 5mL

008425 OPTIGLAZE color Clear HV, 5mL

008425 OPTIGLAZE color Clear HV, 5mL

002613 OPTIGLAZE 15mL



TOUCH *of* GENIUS™

