

# QUICKCHECK INSTRUCTIONS



## CROWNS, BRIDGES, INLAYS AND ONLAYS

- 01 Shake can well before use
- 02a Holding the can desired distance from the restoration, lightly spray enough Quickcheck to cover the entire inside of the restoration (use with or without nozzle extension)
- 02b Holding the can desired distance from the stone die, lightly spray enough Quickcheck to cover the die (use with or without nozzle extension)
- 03 Seat the restoration
- 04 Remove the restoration
- 05 Check inside of restoration for high spot markings or where Quickcheck transferred from the die indicating premature contacts
- 06 Adjust marked area with desired rotary instrument
- 07 Repeat above procedure until high spot markings no longer appear
- 08 Clean Quickcheck from restoration



## PARTIAL FRAMEWORKS

- 01 Shake can well before use
- 02 Holding the can desired distance from metal framework, lightly spray enough Quickcheck to cover the palatal area and the clasps of the framework (use with or without nozzle extension)
- 03 Seat framework
- 04 Remove framework, check for marked areas on the framework and clasps
- 05 Adjust high spot or binding areas
- 06 Repeat above procedure until high spot markings are eliminated
- 07 Clean and polish framework to remove any scratches made by adjustments



## INTERPROXIMAL AREA

- 01 Shake can well before use
- 02 Holding the can desired distance from the interproximal areas of the restoration, lightly spray enough Quickcheck to cover the mesial and distal areas of the restoration (use with or without nozzle extension)
- 03 Seat restoration, checking for proper contact points
- 04 If contact points are too tight, remove crown, adjust high spot marks with desired rotary instrument
- 05 Repeat above procedure until desirable contact points are obtained
- 06 Polish area to remove any scratches made by adjustments



## ADJUSTING DENTURES (WHITE ONLY)

- 01 Shake can well before use
- 02 Holding the can desired distance from the palatal area of the denture, lightly spray White Quickcheck on the entire palatal area or suspected high spot area (use with or without nozzle extension)
- 03 Seat denture, move bite into centric position, remove denture and check for high spot areas
- 04 Adjust any high spot area with denture bur
- 05 Repeat procedure until high spot areas are removed
- 06 Clean denture with water and brush
- 07 Polish and pumice adjusted area



## CAD/CAM SYSTEMS (WHITE ONLY)

- 01 Shake can well before use
- 02 Holding the can desired distance from the die that is to be scanned lightly coat with White Quickcheck (use with or without nozzle extension)
- 03 Scan prepped die